

HeadMouse® Extreme Adjustable Bracket Installation Instructions for DynaVox V and Vmax



The HeadMouse Extreme (“HeadMouse”) mounting bracket for the DynaVox V and Vmax mounts securely on top of the DynaVox device while allowing the HeadMouse to be adjusted vertically, up and down to accommodate the personal seating preference of the user.



Front-view, HeadMouse
Pointing Forward



Side-view, HeadMouse
Rotated Down



Side-view, HeadMouse
Rotated Up

The HeadMouse Extreme Mounting Kit for the Dynavox V and Vmax includes:

- HeadMouse Bracket with Dual-Lock
- Allen wrenches (1-large, 1-small)
- Screws (3-long, 3-short)
- USB cable
- Adhesive cable clasp

Instructions for attaching the HeadMouse bracket to the DynaVox V:

1. Remove all package contents. Separate screws and Allen wrenches.
2. Using large Allen wrench remove the 4 *center* screws, 2 in the front and 2 from the back, from the top of the DynaVox V device.
3. With Dual-Lock attachment facing forward toward the DynaVox V screen-side, align the screw openings of the HeadMouse bracket over the open screw slots on top of the DynaVox V device.
4. Using the small Allen wrench, attach the bracket to the top of the DynaVox V device using 2 long screws in the front and 2 short screws in the back. Screws should be “snug” only, be sure to not over tighten. (*Note: You will use only 2-long and 2-short screws, the remaining screws in the bracket package are extra.*)

5. With the USB connector pointing up, attach the HeadMouse to the bracket front by pressing the Dual-Lock on the back of the HeadMouse to the Dual-Lock on the bracket. Press together firmly.
6. Connect small end of USB cable to USB port on HeadMouse Extreme.
7. Connect flat end of USB cable to USB port on the back of the DynaVox V device.
8. Remove the adhesive tape cover from the cable clasp.
9. With the cable clasp opening-side facing out, affix the cable clasp on the back of the DynaVox V device directly below the USB cable port.
10. Open the cable clasp.
11. Fold the USB cable gently into the cable clasp and close.
12. Return Allen wrenches and extra screws to the bracket package for safe keeping.



Instructions to remove the HeadMouse Extreme from mounting bracket:

1. Disconnect the USB cable.
2. Gently grasp the HeadMouse and rotate the mount and HeadMouse back until the HeadMouse is pointing up.
3. Then continue rotating the HeadMouse until the Dual-lock releases.
4. This technique minimizes stress on the Dual-lock, bracket, fasteners and Dynavox V case.



Customer Support:

If you have questions or need further assistance please contact Customer Support at (972) 606-8740 or support@orin.com.

Remember to shop our online store at: <http://shop.orin.com>



Origin Instruments Corporation
 854 Greenview Dr
 Grand Prairie, TX 75050
 Tel: 972-606-8740, Fax: 972-606-8741
 Email: support@orin.com
 Web: www.orin.com
 Online Store: <http://shop.orin.com>

© 2007 Copyright by Origin Instruments Corporation. All rights reserved. HeadMouse is registered trademark of Origin Instruments Corporation. SofType, Swifty, Dragger, Beam and AutoClick are trademarks of Origin Instruments Corporation. All other marks are property of their respective owners.